

Tri-bevel, ultra-sharp, electropolished and lubricated needle

for patient comfort



Thin-Wall Technology Canula •

provides for optimal flow and higher tube fill-rates



Ergonomic wing design

permits improved positioning during application



enables one-hand activation of safety mechanism, audible click confirms that needle is securely locked



for better visibility of collection tubes filling up with blood, tube holders provided separately or pre-attached

 Multi-sampling, universal Luer

allows for tight fit to tube holders and multiple blood sample collection from one patient

■ DEPF-free, low memory and flexible tubing


> assists in handling and preserves the blood chemical and physiological composition

WWW.ONE-CARE.COM






Remove the needle protective cover.

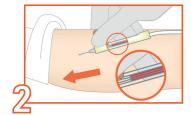

Hold the wings together and insert needle bevel side up into the patient's vein. Observe a flash of blood in the transparent chamber, indicating the vein has ben accessed. Collect blood sample according to your facility's policy.

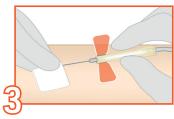

Gently remove the needle from vein.



Glide the yellow safety shield forward and pull tubing backwards with the other hand until an audible "Click" is heard, which indicates that the needle has been permanently locked.



Dispose of all used materials as per your facility's policy.



ORDERING INFORMATION

Available in different needle configurations and with/without pre-attached tube holders:

	CODE	DESCRIPTION - NEEDLE (DIAMETER x LENGTH)	COLOR	PACKAGING
1	5040	ONE-CARE® GLIDE & LOCK Safety Blood Collection Winged Sets 25G x 3/4", Luer-Adapter, 12" Tube		50 pcs / box 20 bx (1000 pcs) / case
	5041	ONE-CARE® GLIDE & LOCK Safety Blood Collection Winged Sets 24G x 3/4", Luer-Adapter, 12" Tube		50 pcs / box 20 bx (1000 pcs) / case
	5042	ONE-CARE® GLIDE & LOCK Safety Blood Collection Winged Sets 23G x 3/4", Luer-Adapter, 12" Tube		50 pcs / box 20 bx (1000 pcs) / case
	5043	ONE-CARE® GLIDE & LOCK Safety Blood Collection Winged Sets 22G x 3/4", Luer-Adapter, 12" Tube		50 pcs / box 20 bx (1000 pcs) / case
	5044	ONE-CARE® GLIDE & LOCK Safety Blood Collection Winged Sets 21G x 3/4", Luer-Adapter, 12" Tube		50 pcs / box 20 bx (1000 pcs) / case
	5045	ONE-CARE® GLIDE & LOCK Safety Blood Collection Winged Sets 20G x 3/4", Luer-Adapter, 12" Tube		50 pcs / box 20 bx (1000 pcs) / case
	5046	ONE-CARE® GLIDE & LOCK Safety Blood Collection Winged Sets 19G x 3/4", Luer-Adapter, 12" Tube		50 pcs / box 20 bx (1000 pcs) / case
				25 nos / how
	5040-H	$ONE-CARE^{\textcircled{\tiny{\textbf{B}}}} \ GLIDE \ \& \ LOCK Safety \ Blood \ Collection \ Winged \ Sets \ 25G \times 3/4", Pre-Attached \ Holder, 12" Tube$		25 pcs / box 20 bx (500 pcs) / case
	5041-H	$ONE-CARE^{\circledcirc}GLIDE\&LOCKSafetyBloodCollectionWingedSets24Gx3/4", Pre-AttachedHolder, 12"Tube$		25 pcs / box 20 bx (500 pcs) / case
	5042-H	$ONE-CARE^{\textcircled{\tiny{0}}}\ GLIDE\ \&\ LOCK\ Safety\ Blood\ Collection\ Winged\ Sets\ 23G\ x\ 3/4",\ Pre-Attached\ Holder,\ 12"\ Tube$		25 pcs / box 20 bx (500 pcs) / case
	5043-H	$ONE-CARE@\ GLIDE\ \&\ LOCK\ Safety\ Blood\ Collection\ Winged\ Sets\ 22G\ x\ 3/4"\ , Pre-Attached\ Holder\ , 12"\ Tube$		25 pcs / box 20 bx (500 pcs) / case
	5044-H	${\tt ONE-CARE}^{\small{\textcircled{\tiny \$}}}{\tt GLIDE}\&{\tt LOCKSafety}{\tt Blood}{\tt Collection}{\tt Winged}{\tt Sets}{\tt 21G}x3/4", {\tt Pre-Attached}{\tt Holder}, {\tt 12"}{\tt Tube}$		25 pcs / box 20 bx (500 pcs) / case
	5045-H	$ONE-CARE^{\small \circledcirc}GLIDE\&LOCKSafetyBloodCollectionWingedSets20Gx3/4", Pre-AttachedHolder, 12"Tube$		25 pcs / box 20 bx (500 pcs) / case

WWW.ONE-CARE.COM

ONE-CARE® GLIDE & LOCK Safety Blood Collection Winged Sets 19G x 3/4", Pre-Attached Holder, 12" Tube



5046-H



25 pcs / box

20 bx (500 pcs) / case