

ONE-CARE™

Insulin Syringes



Insulin delivery has never been comfortable...

Gentle

Precise needle grinding and special lubrication for gentle injection



Convenient

Wide flaps for convenient finger recess

Precise

Precise graduation and marking for accurate dosing. Reduce "dead space" to prevent medication loss

CONVENIENT

- Wide flaps for easy and stable hold
- Smooth move of the plunger for easy injection

GENTLE

- Tri-beveled, electro-polished cannula needle coated with medical grade lubricant for user comfort
- Thin wall technology with small needle diameter versions available for delicate injections

PRECISE

- Precise graduation and marking for accurate dosing
- High quality molded components and rubber stopper for reduced dead space to prevent medication loss

VERSATILE

- Different sizes 0.3 - 0.5 - 1 ml for right applications
- Multiple needle configurations 29G - 31 G


INSTRUCTIONS FOR USE

■ Intuitive and easy to follow steps for user comfort.




Step 1

Always wash your hands

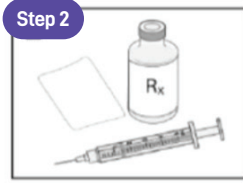


Step 2

Get all of your supplies

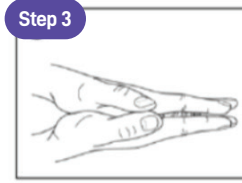


Step 3

Roll the insulin vial between your hands, if using a cloudy insulin

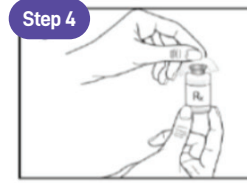


Step 4

Wipe the top of the vial with an alcohol swab

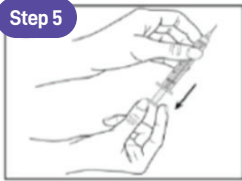


Step 5

Pull the syringe plunger down to the desired units

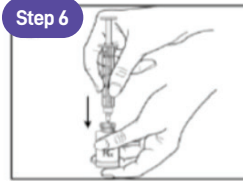


Step 6

Push the syringe needle into the vial

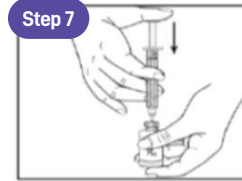


Step 7

Push the plunger down

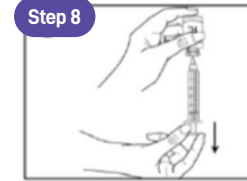


Step 8

Turn the vial upside down. Pull the plunger down to withdraw insulin

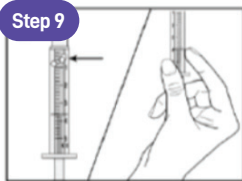

Step 9

Remove all the air bubbles. Check to see if your dose is correct.

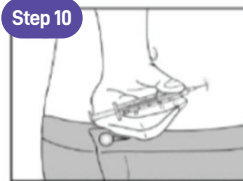

Step 10

Pick a clean injection site. Push the needle into the skin.



Step 11

Push the plunger in. Pull the needle out of the skin.

ORDERING INFORMATION

INSULIN U40		29G x 1/2"	12.7 mm
TUBERCULIN		29G x 1/2"	12.7 mm
		Luer	
INSULIN U100	1 ml	27G x 1/2"	12.7 mm
		29G x 5/16"	8 mm
		29G x 1/2"	12.7 mm
		30G x 1/2"	12.7 mm
		30G x 5/16"	8 mm
		31G x 4/16"	6 mm
		Luer	
	0.5 ml	29G x 1/2"	12.7 mm
		30G x 1/2"	12.7 mm
		31G x 5/16"	8 mm
31G x 4/16"		6 mm	
0.3 ml	29G x 1/2"	12.7 mm	
	30G x 1/2"	12.7 mm	
	31G x 5/16"	8 mm	
	31G x 4/16"	6 mm	



WWW.ONE-CARE.COM

Americas

USA
MediVena
1 Glenlake Parkway, Ste 700
Atlanta, GA 30328, USA
+1 404 422 1513
info@medivena.com
www.medivena.com

Europe

Poland
MediVena Sp. z o.o.
Al. Jana Pawła II 27
00-867 Warsaw, Poland
+48 22 299 44 19
info@medivena.com
www.medivena.com

